

Aillon Tube Benders Sterilization Guidelines (Anesthesia Associates)

Cleaning:

Parts may be washed in warm, soapy water (composed of mild detergents or a natural liquid soap) and carefully scrubbed with a soft brush, rinsed and dried.

- 1. Gas Sterilization: Ethylene Oxide Gas Sterilization using 100% concentration for 8.5 hours at 120°F (48° C), with aeration of 48 hours at 90° to 130° F (32° to 54° C).**
- 2. Cold Sterilization: Liquid “Cold” Sterilization using a 2% Gluteraldehyde solution (ie: Cidex or other) with soak time of 10 hours at 59° to 86° F (15° to 30° C).**
- 3. Steam Sterilization: Gravity Steam Sterilization (autoclave) using a 45-minute exposure time at 250° F (121° C).**