

# GUIDELINES for PRODUCT CLEANING and STERILIZATION

Part Number(s): 00-436-06 thru 00-436-12 & E-210-xx, E-215-xx, E-220-xx

Part Description(s): Semi-Clear Flex Connectors & Breathing Tubes Latex Free

**Warning:** The indicated sterilization / high-level-disinfection guidelines have been determined from sterilization efficacy studies performed on this or a related product and may be limited by the process's compatibility with the product's specific materials of construction. Process should be performed following the parameters indicated, using approved hospital/contractor procedures. The hospital/contractor is responsible for validating all protocols and verifying the results of each sterilization process.

## **CLEANING GUIDELINES for this product(s):**

- CLEAN-1 (Wash):** Parts may be washed in warm, soapy water (composed of mild detergents or a natural liquid soap) and carefully scrubbed with a soft brush, rinsed and dried.
- CLEAN-2 (Wipe):** Parts may be cleaned with an alcohol wipe, a disinfectant wipe, or by sponging with a clean damp soft cloth, and dried.
- CLEAN-3 (Insert):** See manufacturer's product package/insert for specific details.
- NOTES :** \_\_\_\_\_  
\_\_\_\_\_

## **STERILIZATION / HIGH-LEVEL-DISINFECTION GUIDELINES for this product(s):**

- STERILE-1 (ETO) :** Ethylene Oxide Gas Sterilization using 100% concentration for 8.5 hours at 120° F (48° C), with aeration of 48 hours at 90° to 130° F (32° to 54° C).
- STERILE-2 (Cold) :** Liquid "Cold" Sterilization using a 2% Gluteraldehyde solution (ie: Cidex or other) with soak time of 10 hours at 59° to 86° F (15° to 30° C).
- STERILE-3 (Autoclave) :** Gravity Steam Sterilization (autoclave) using a 45-minute exposure time at 250° F (121° C).
- STERILE-4 (Insert) :** See attached manufacturer's product package/insert for specific sterilization information details.
- STERILE-5 (Pasteurize) :** High Level Disinfect with Washer Pasteurizer, not less than 30min at 160° to 170° F (71° to 77° C), remove all air pockets before initiating cycle.
- NOTES :** \_\_\_\_\_  
\_\_\_\_\_

## **OTHER RELATED INFORMATION for this product:**

- NOTES :** \_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

FORM GENERATED BY: CB DATE: 9/12/18 REV: C

00-436-xx, E-210-xx, E-215-xx, E-220-xx 00-436-xx, E-210-xx, E-215-xx, E-220-xx Cleaning & Sterilization