

## Peep Valves Cleaning and Sterilization Guidelines (SunMed)

**Warning:** The indicated sterilization guidelines have been determined from sterilization efficacy studies performed on this or a related product and may be limited by the process's compatibility with the product's specific materials of construction. Sterilization should be performed following the parameters indicated, using approved hospital/contractor procedures. The hospital/contractor is responsible for validating all protocols and verifying the results of each sterilization process. All components should be inspected prior to use.

**CAUTION:** Do not disassemble the PEEP Valve. Damage on these parts may occur when disassembling them.

**CLEANING GUIDELINES for the product: CLEAN-1 (Wash):** Parts may be washed in warm, soapy water (composed of mild detergents or a natural liquid soap) and carefully scrubbed with a soft brush, rinsed and dried.

**STERILIZATION GUIDELINES for this product: STERILE-1 (Autoclave):** Gravity Steam Sterilization (autoclave) using a 30-minute exposure time a 230°F (110°C) and 10 minute dry time.